



We're proud to call this finishing & polishing system ours.

Enhance[®]
Finishing System

PoGo[®]
One Step Diamond Micro-Polisher



"Enhance is the only finishing system to be a two-time winner of products evaluators can't live without."

Independent Research Institute (July 2004)

In a recent clinical study of the "Effect of finishing and polishing procedures on surface roughness, gloss, and color of resin-based composites", PoGo[®] One Step Diamond Micro-Polishers provided both the highest gloss and smoothest surface of the market leading systems tested.¹

We think you'll be proud to call it yours.

DENTSPLY Caulk continues to provide restorative solutions that make your life easier!

For finishing and polishing composites, the combination of the **Enhance Finishing System** and the **PoGo One Step Diamond Micro-Polishers** creates an easy, dependable method for achieving natural looking, high luster restorations.



Clinical Benefits

Enhance Finishing System

- **One Component:** Why struggle with four when you can Enhance with one.
- **Control:** Intermediate and final finishing is completed with the Enhance Finishing System. The pressure applied controls the aggressiveness. More pressure for an aggressive initial finish; less pressure to smooth the surface.


PoGo One Step Diamond Micro-Polishers

- **One Component:** Single-step diamond polishers, used with light pressure for a beautiful luster.
- **Fast:** Short technique time (40-60 seconds) is all that is needed to achieve a natural, long-lasting polish on any composite material.
- **Simple Technique:** After finishing with Enhance, polish with PoGo Diamond Micro-Polishers to create ultimate luster. Start with a light pressure and decrease pressure to a “feather-like” touch to bring out the highest luster.

Easy, Fast Technique




Comparison of Sof-Lex™ Superfine polishing disc to PoGo Diamond Micro-Polisher on human enamel for 30 seconds at 1000x.

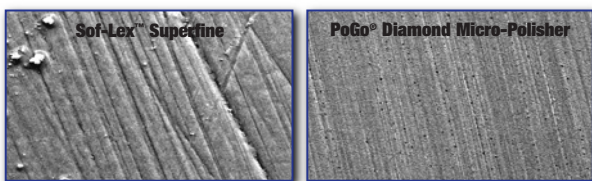

Roughness Comparison

Effect of Final Step on Surface (by Material with Discs)

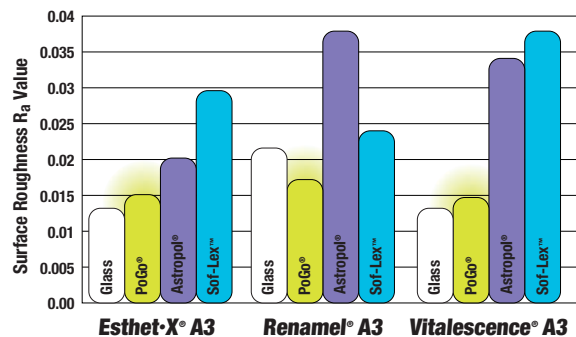

Why PoGo your restoration?

PoGo Diamond Micro-Polishers give you a higher luster on different restorative materials.² “Glass” refers to the smoothness of composite cured against a piece of glass. The graph shows the smoothness after utilizing the last component of each polishing system.

Ordering Information

ENHANCE® Finishing and Polishing System

- 624045 Finishing Disc Refill (40 per refill)
- 624055 Finishing Cup Refill (40 per refill)
- 624065 Finishing Point Refill (40 per refill)
- 631400 Prisma®•Gloss™ Composite Polishing Paste 1 Syringe - 4 g
- 631450 Prisma®•Gloss™ Extrafine Composite Polishing Paste 1 Syringe - 4 g
- 624020 Polishing Cups Refill 100 Cups & 1 Mandrel

POGO® One Step Diamond Micro-Polisher

- 662020 Enhance® & PoGo® Complete Kit Enhance Finishers - 10 each of Discs, Cups, & Points PoGo Polishers - 10 each of Discs, Cups, & Points
- 662010 Disc Refill Package (40 per refill)
- 662022 Points Refill Package (40 per refill)
- 662023 Cups Refill Package (40 per refill)

¹ R. Paravina, L. Roeder, H. Lu, K. Vogel, & J. Powers “Effect of finishing and polishing procedures on surface roughness, gloss, and color of resin-based composites.” American Journal of Dentistry, Vol. 17, No. 4, August 2004.

² Data on file.

Astropol, Sof-Lex, Renamel and Vitalescence are not registered trademarks of DENTSPLY International. Clinical photography by Dr. Jeff Blank