

# Esthet•X®

micro matrix restorative

Opacious  
Dentin  
Shades

Regular  
Body  
Shades

Translucent  
Enamel  
Shades

## Class IV *Directions For Use*

DENSPLY  
CAULK



1. Condition enamel for at least 15 seconds and dentin for 15 seconds or less. Rinse for 10 seconds. Blot dry but keep moist.




2. **A.** Apply Prime & Bond® NT™ Adhesive to the preparation and saturate all surfaces for at least 20 seconds.  
**B.** Gently air dry for 5 seconds. Surfaces should have a uniform, glossy appearance. **Light cure for 10 seconds.** If not glossy, repeat steps 2A and 2B.



3. Apply Esthet•X® Micro Matrix Restorative Opacious Shade up to 2mm (e.g., A2-0) to mask the darkness of the mouth or discolored dentition. **Light cure for 20 seconds.**




4. Apply Esthet•X Micro Matrix Restorative Regular Body Shade up to 2mm (e.g., A2). **Light cure for 20 seconds.**

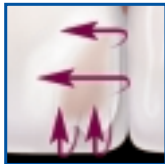

5. Apply Esthet•X Micro Matrix Restorative Translucent Enamel Shade up to 2mm (e.g., CE/Clear Enamel). This translucent layer will replace incisal edges and add vitality to the restoration. **Light cure for 20 seconds.**



6. **A.** Define the final anatomy with a multi-fluted bur or a diamond finishing bur.  
**B.** Continue finishing with Enhance® Discs, Cups, or Points.

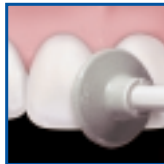

7. For a glass-like luster, polish the restoration with the PoGo® One Step Diamond Micro-Polisher System and/or Prisma® Gloss™ Polishing Pastes.

# Esthet•X<sup>®</sup>

micro matrix restorative

## Class II *Directions For Use*



- 1. A.** Round internal forms and refine margins, then rinse.  
**B.** Place matrix band. **C.** Apply Caulk<sup>®</sup> 34% Tooth Conditioner Gel to both the enamel and dentin for 15 seconds. Rinse for 10 seconds. Blot dry until moist and glistening.



- 2. A.** Apply Prime & Bond<sup>™</sup> NT<sup>™</sup> Adhesive to the preparation and saturate all surfaces for at least 20 seconds. **B.** Remove excess solvent by gently air drying for 5 seconds. **Light cure for 10 seconds.** If not glossy, repeat steps 2A and 2B.



**OPTIONAL**

- 3. A.** Place a 0.5 – 1mm layer of either Esthet•X<sup>®</sup> Flow Liquid Micro Hybrid Material or Dyract<sup>®</sup> flow Compomer. Both materials are indicated for a Class II liner. **B.** For the Class II liner indication, all shades should be light cured for **40 seconds.**



- 4.** Place increments up to 2mm, with Esthet•X Restorative Body or Opaque Shade, **light curing each increment 20 seconds.** Fill to the dentin-enamel junction.



- 5.** The Esthet•X Translucent Enamel Shade is placed as a final layer (up to 2mm). Sculpt and contour. **Light cure for 20 seconds.**



- 6.** Define the final anatomy with a multi-fluted bur or a diamond finishing bur. **A.** Continue finishing with Enhance<sup>®</sup> Discs, Cups, or Points. **B.** Polish the restoration with PoGo<sup>®</sup> Discs, Cups, and Points and/or Prisma<sup>®</sup> Gloss<sup>™</sup> Polishing Pastes.

Esthet•X<sup>®</sup>  
Regular Body  
or Opaque  
Dentin Shades

Esthet•X<sup>®</sup>  
Translucent  
Enamel  
Shades

DENTSPLY  
CAULK

DENTSPLY Caulk  
38 West Clarke Avenue  
P.O. Box 359  
Milford, DE 19963-0359  
www.dentsply.com