

Xeno[®] IV

One Component Light Cured Self-Etching Dental Adhesive

Technique Guide

35°-46°F
2°-8°C

Storage Temperature



Consult directions
prior to use.



DOSAGE
Dispense
2-3 drops of
liquid into a
clean mixing
well.


DENTSPLY
CAULK




*For larger restorations,
two (2) 20-second
scrubbing applications
are recommended.

#568002 (7/23/04)

Apply generous amounts
onto cavity surfaces. Actively
scrub for 15 sec. * **Repeat.**

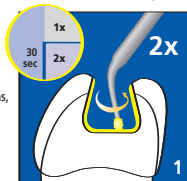

Uniformly spread adhesive by
a gentle stream of air pressure
until there is no more flow.

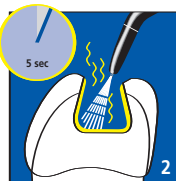

Light cure for at least
10 sec. Place restoration
immediately.

