

Troubleshooting Information




Incorrect

Need to have more contact on adjacent teeth

Correct

Occluding contacts evident

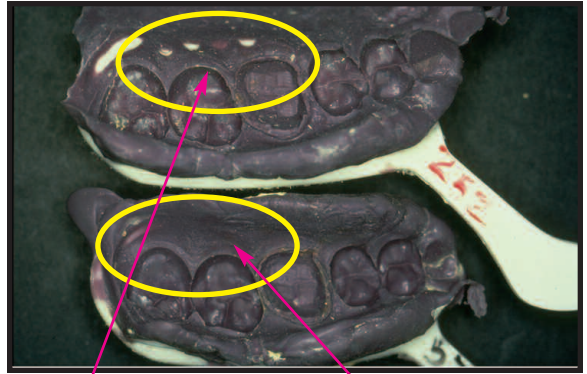

Incorrect

Correct

INADEQUATE OCCLUSION WITH CLOSED-BITE IMPRESSION

Take closed bite impression and hold it to a light.

TISSUE CONTACT WITH TRAY

Problem:

- Size and shape of tray is not correct
- Tray is not seated correctly
- Not enough tray or wash material used

Solution:

- Fill tray 2/3 full. Use more wash material around preparation and adjacent teeth.
- Use custom tray or 2-step putty/wash in a stock tray.
- Check tray size and retake impression.
- Seat tray evenly.
- Avoid tooth contact with tray while seating.